Purefill® PUREFILL

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Features

- Indicated for embolization of arteriovenous malformations.
- Liquid embolic implant, sterile, translucent, with low viscosity composed of **n-hexyl cyanoacrylate monomers**.
- Apply in solution with a contrast liquid mixture.
- Fast polymerization when the liquid embolic agent comes in contact with tissue or moist environments (blood/water).
Once polymerisation is completed the glue has no more adhesive power.

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Instructions for use

- Remove the blister pack from its storage place a few minutes before use to adjust it to room temperature.
- Verify the fluidity and transparency of the product.
- For all occlusion procedures, mix PUREFILL® with iodized oil (such as Lipiodolultrafluid®) in 01:02 proportions in an appropriate zone : a sterile glass beaker is recommended.
- Homogenize, then withdraw the mixture with a luer lock syringe for injection of 1 ml to 3 ml.
- Position the micro-catheter according to the arterial embolization radiological procedure after rinsing with a 5% glucose solution.
- Connect the syringe to the micro-catheter and inject the mixture manually under radiographic monitoring.
- After injection, rapidly remove the micro-catheter to prevent adhesion of the tip.
- After each injection, discard the micro-catheter, all the ancillary articles and the remaining mixture.

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