DARVIN LOC PERMANENT

CE 0373

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INSTRUCTIONS FOR USE

Description:

Darvin Loc Permanent suture is a sterile non-absorbable surgical suture device of synthetic origin produced from Polypropylene. The suture is a monofilament undyed or dyed blue (C.I. 74160 phthalocyanine) and is available in a range of gauges sizes (USP/EP) and lengths, non-needled or attached to stainless steel needles of different size and shapes and is barbed.

Indications:

Darvin Loc Permanent sutures are intended for use in all surgical procedures, at the user's discretion.

Properties:

The Darvin Loc Permanent is non-absorbable, or subject to degradation by the action of enzymes and it maintains its own tensile strength in vivo as well after years from implantation..

Precautions:

Darvin Loc Permanent sutures are to be used in operating theatres, nursing homes and outpatient clinics or in any hospital environment, by qualified personnel, experienced in technical and surgical procedures. Care should be taken when handling the suture material to avoid any bending or flattening to the thread due to the application of surgical instruments such as forceps or needleholders.

Surgical sutures are disposable medical device and must be disposed of as hospital waste material.

Adverse reactions:

Darvin Loc Permanent suture may be cause local inflammatory reaction.

Warnings:

- Disposable
- Do not use if package is damaged
- Use by expiry date
- Keep in cool dry place, away from any direct heat

Sterility:

- Darvin Loc Permanent sutures are sterilized by Ethylene Oxyde
- Sterile unless package is opened or damaged
- Do not re-sterilize

Symbols used on labelling:

= Do not reuse

= Manufactured by Year and Month

= Use by Year and Month

= Manufacturer

= Sterile unless package is opened or damaged. Sterilization Method: Ethylene Oxyde

= CE mark and identification number of the Notified body. Product conforms to the essential requirements of Directives 93/42/EEC and 2007/47/EC concerning Medical Devices

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