	Rx Only		Non-Sterile	
	Latex Free		Prescription Only	

INDICATIONS FOR USE
The Cygnet clamp is to be used with INTRACK® inserts as indicated for surgical clamping to temporarily occlude blood vessels in cardiovascular, peripheral vascular and general vascular applications. The Cygnet clamp is available in 33mm, 60mm and 80mm insert lengths, which are compatible with INTRACK®-33mm, 60mm and 80mm inserts.

DESCRIPTION
The Cygnet clamp has a telescopic retractable rigid shaft that allows the user to insert the clamp to the desired open vessel sizes and where space is limited. When the rigid shaft is retracted, the inner flexible shaft enables the clamp handle to be placed out of the way, facilitating access to the surgical site. The flexible shaft consists of 40 surgical wires, which allows the flexible shaft to be rotated away from the operative site.

CAUTION: The Cygnet clamp must be loaded with sterile INTRACK® inserts prior to use.

CAUTION: The Cygnet clamp is capable of providing large clamping forces that are necessary in some surgical procedures. Do not apply pressure that is too far anteriorly or posteriorly to prevent tissue damage.

CAUTION: Care must be taken when clamping diseased or damaged vessels. As with any cardiovascular clamp, avoid excessive clamping force that would damage the vessel.

Inspection:

The clamp should be inspected carefully prior to each use and during the cleaning procedure to identify any signs of unusual wear, cracking or corrosion. Microscopic cracks may be detected by appearance of rust on the instrument. Inspect the inner cable for fraying, fraying or any visible damage to the outer cable. The inner cable can be exposed by gently pulling on the handle while the handle fully opens. Should any irregularity be noted, the clamp should not be used.

Placement of INTRACK® inserts:

Ensure the Cygnet clamp is clean and sterile before use.

1. Hold the clamp housing and release the handle retainer.

2. Hold the jaw housing and turn the jaws sideways.

3. Place the angled edge of the tip of the insert directly into the channel using the thumb and forefinger of your free hand (see Fig. 2).

4. Once the desired vessel is securely clamped, the rigid shaft may be retracted via the finger grip collar, exposing the flexible shaft (see Fig. 5).

5. Insert the second jaw insert by repeating steps 2-5.

Clamp Operation:

1. Ensure the clamp is clean and sterile before use.

2. Release the handle retainer and release the handle retainer.

3. Hold the jaw housing and turn the jaws sideways.

4. Place the angled edge of the tip of the insert directly into the channel using the thumb and forefinger of your free hand (see Fig. 2).

5. Once the desired vessel is securely clamped, the rigid shaft may be retracted via the finger grip collar, exposing the flexible shaft (see Fig. 5).

6. Insert the second jaw insert by repeating steps 2-5.

Removal of Inserts:

The INTRACK® inserts are disposable and must be removed before cleaning or sterilizing the Cygnet clamp.

1. Ensure the clamp is clean and sterile before use.

2. Slide the inserts out of the jaw channel and properly dispose of the contaminated used inserts.

Cleaning:

CAUTION: Vitalec recommends the use of enzymatic detergents for optimum cleaning results.

The use of detergents containing potassium hydroxide or sodium hydroxide is not recommended as detergents containing these chemicals may cause instrument discolouration.

CAUTION: It is not recommended to use temperatures in excess of 270 °F (132 °C) for cleaning, drying and sterilization. Excessively high temperatures may cause instrument discolouration.

During the cleaning process, ensure that the jaws are open and the ratchets are released.

1. Turn the clamp housing and turn the jaws sideways.

2. Turn the clamp housing and turn the jaws sideways.

3. Turn the clamp housing and turn the jaws sideways.

4. Once the desired vessel is securely clamped, the rigid shaft may be retracted via the finger grip collar, exposing the flexible shaft (see Fig. 5).

5. Turn the clamp housing and turn the jaws sideways.

6. Turn the clamp housing and turn the jaws sideways.

7. Turn the clamp housing and turn the jaws sideways.

8. Turn the clamp housing and turn the jaws sideways.

9. Turn the clamp housing and turn the jaws sideways.

10. Turn the clamp housing and turn the jaws sideways.

11. Turn the clamp housing and turn the jaws sideways.

12. Turn the clamp housing and turn the jaws sideways.

13. Turn the clamp housing and turn the jaws sideways.

14. Turn the clamp housing and turn the jaws sideways.

15. Turn the clamp housing and turn the jaws sideways.

16. Turn the clamp housing and turn the jaws sideways.

17. Turn the clamp housing and turn the jaws sideways.

18. Turn the clamp housing and turn the jaws sideways.

19. Turn the clamp housing and turn the jaws sideways.

20. Turn the clamp housing and turn the jaws sideways.

21. Turn the clamp housing and turn the jaws sideways.

22. Turn the clamp housing and turn the jaws sideways.

23. Turn the clamp housing and turn the jaws sideways.

24. Turn the clamp housing and turn the jaws sideways.

25. Turn the clamp housing and turn the jaws sideways.

26. Turn the clamp housing and turn the jaws sideways.

27. Turn the clamp housing and turn the jaws sideways.

28. Turn the clamp housing and turn the jaws sideways.

29. Turn the clamp housing and turn the jaws sideways.

30. Turn the clamp housing and turn the jaws sideways.

31. Turn the clamp housing and turn the jaws sideways.

32. Turn the clamp housing and turn the jaws sideways.

33. Turn the clamp housing and turn the jaws sideways.

34. Turn the clamp housing and turn the jaws sideways.

35. Turn the clamp housing and turn the jaws sideways.

36. Turn the clamp housing and turn the jaws sideways.

37. Turn the clamp housing and turn the jaws sideways.

38. Turn the clamp housing and turn the jaws sideways.

39. Turn the clamp housing and turn the jaws sideways.

40. Turn the clamp housing and turn the jaws sideways.

41. Turn the clamp housing and turn the jaws sideways.

42. Turn the clamp housing and turn the jaws sideways.

43. Turn the clamp housing and turn the jaws sideways.

44. Turn the clamp housing and turn the jaws sideways.

45. Turn the clamp housing and turn the jaws sideways.

46. Turn the clamp housing and turn the jaws sideways.

47. Turn the clamp housing and turn the jaws sideways.

48. Turn the clamp housing and turn the jaws sideways.

49. Turn the clamp housing and turn the jaws sideways.

50. Turn the clamp housing and turn the jaws sideways.

51. Turn the clamp housing and turn the jaws sideways.

52. Turn the clamp housing and turn the jaws sideways.

53. Turn the clamp housing and turn the jaws sideways.

54. Turn the clamp housing and turn the jaws sideways.

55. Turn the clamp housing and turn the jaws sideways.

56. Turn the clamp housing and turn the jaws sideways.

57. Turn the clamp housing and turn the jaws sideways.

58. Turn the clamp housing and turn the jaws sideways.

59. Turn the clamp housing and turn the jaws sideways.

60. Turn the clamp housing and turn the jaws sideways.

61. Turn the clamp housing and turn the jaws sideways.

62. Turn the clamp housing and turn the jaws sideways.

63. Turn the clamp housing and turn the jaws sideways.

64. Turn the clamp housing and turn the jaws sideways.

65. Turn the clamp housing and turn the jaws sideways.

66. Turn the clamp housing and turn the jaws sideways.

67. Turn the clamp housing and turn the jaws sideways.

68. Turn the clamp housing and turn the jaws sideways.

69. Turn the clamp housing and turn the jaws sideways.

70. Turn the clamp housing and turn the jaws sideways.

71. Turn the clamp housing and turn the jaws sideways.

72. Turn the clamp housing and turn the jaws sideways.

73. Turn the clamp housing and turn the jaws sideways.

74. Turn the clamp housing and turn the jaws sideways.

75. Turn the clamp housing and turn the jaws sideways.

76. Turn the clamp housing and turn the jaws sideways.

77. Turn the clamp housing and turn the jaws sideways.

78. Turn the clamp housing and turn the jaws sideways.

79. Turn the clamp housing and turn the jaws sideways.

80. Turn the clamp housing and turn the jaws sideways.

81. Turn the clamp housing and turn the jaws sideways.

82. Turn the clamp housing and turn the jaws sideways.

83. Turn the clamp housing and turn the jaws sideways.

84. Turn the clamp housing and turn the jaws sideways.

85. Turn the clamp housing and turn the jaws sideways.

86. Turn the clamp housing and turn the jaws sideways.

87. Turn the clamp housing and turn the jaws sideways.

88. Turn the clamp housing and turn the jaws sideways.

89. Turn the clamp housing and turn the jaws sideways.

90. Turn the clamp housing and turn the jaws sideways.

91. Turn the clamp housing and turn the jaws sideways.

92. Turn the clamp housing and turn the jaws sideways.

93. Turn the clamp housing and turn the jaws sideways.

94. Turn the clamp housing and turn the jaws sideways.

95. Turn the clamp housing and turn the jaws sideways.

96. Turn the clamp housing and turn the jaws sideways.

97. Turn the clamp housing and turn the jaws sideways.

98. Turn the clamp housing and turn the jaws sideways.

99. Turn the clamp housing and turn the jaws sideways.

100. Turn the clamp housing and turn the jaws sideways.

101. Turn the clamp housing and turn the jaws sideways.

102. Turn the clamp housing and turn the jaws sideways.

103. Turn the clamp housing and turn the jaws sideways.

104. Turn the clamp housing and turn the jaws sideways.

105. Turn the clamp housing and turn the jaws sideways.

106. Turn the clamp housing and turn the jaws sideways.

107. Turn the clamp housing and turn the jaws sideways.

108. Turn the clamp housing and turn the jaws sideways.

109. Turn the clamp housing and turn the jaws sideways.

110. Turn the clamp housing and turn the jaws sideways.

111. Turn the clamp housing and turn the jaws sideways.

112. Turn the clamp housing and turn the jaws sideways.

113. Turn the clamp housing and turn the jaws sideways.

114. Turn the clamp housing and turn the jaws sideways.

115. Turn the clamp housing and turn the jaws sideways.

116. Turn the clamp housing and turn the jaws sideways.

117. Turn the clamp housing and turn the jaws sideways.

118. Turn the clamp housing and turn the jaws sideways.

119. Turn the clamp housing and turn the jaws sideways.

120. Turn the clamp housing and turn the jaws sideways.

121. Turn the clamp housing and turn the jaws sideways.

122. Turn the clamp housing and turn the jaws sideways.

123. Turn the clamp housing and turn the jaws sideways.

124. Turn the clamp housing and turn the jaws sideways.

125. Turn the clamp housing and turn the jaws sideways.

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